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
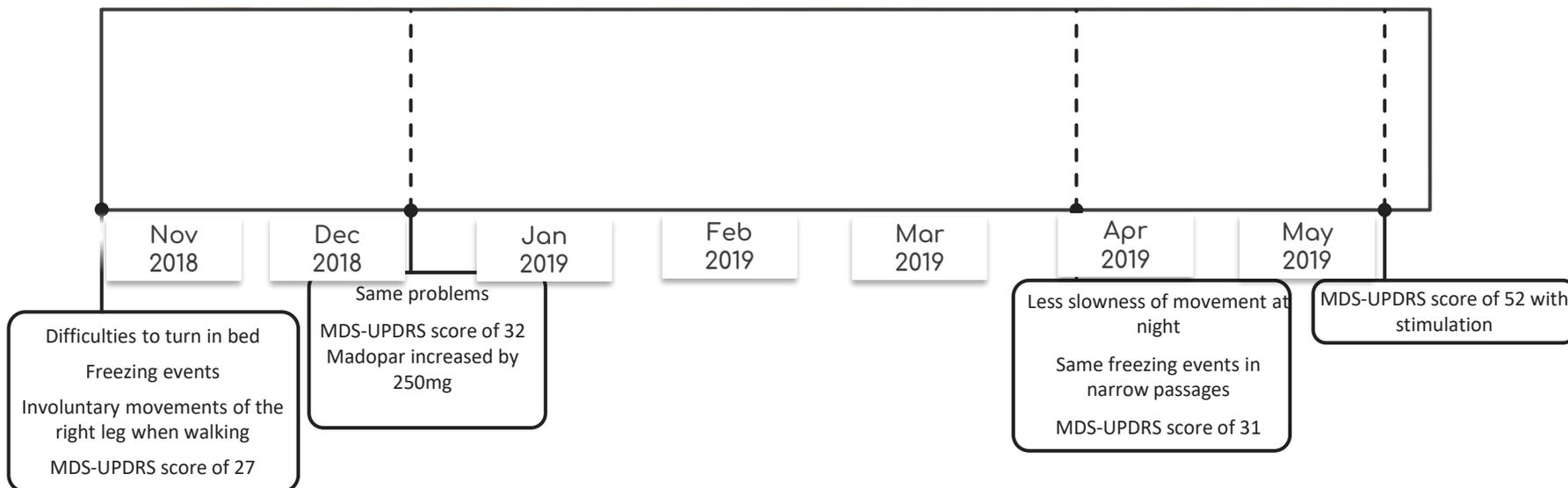
Awakening rate

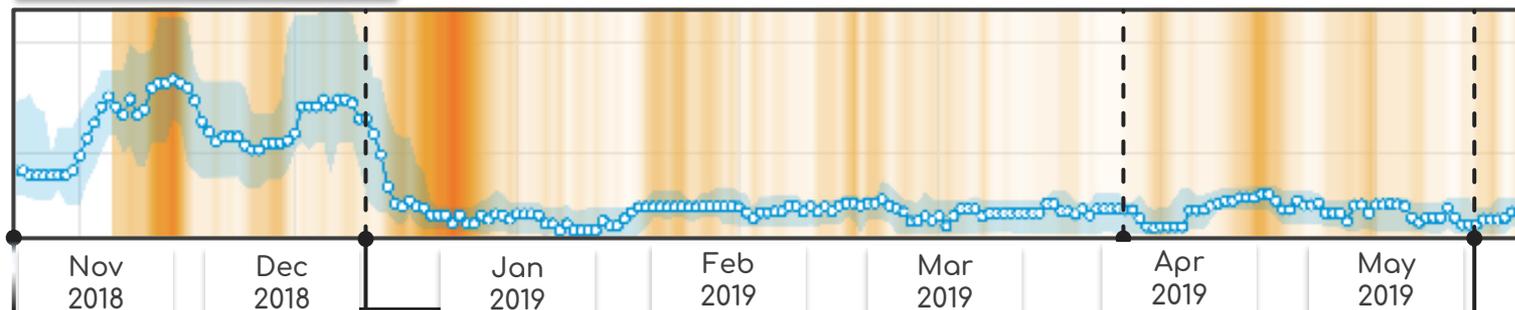Median

Difficulties to turn in bed
Freezing events
Involuntary movements of the right leg when walking
MDS-UPDRS score of 27

Same problems
MDS-UPDRS score of 32
Madopar increased by 250mg

Automatic, continuous measurements of key biomarkers over months, with AI-powered change detection

Less slowness of movement at night
Same freezing events in narrow passages
MDS-UPDRS score of 31

MDS-UPDRS score of 52 with stimulation

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Dr. Julien Bally
Movement disorder specialist

Prof. Philipp Ryvlin
Head of the Dept. of Clinical Neurosciences

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
Continuous Nocturnal Monitoring Parkinson's Disease

DOMO's Clinical Decision Support Engine

Patient ID: 1
Aggregation timescale: 14 days

domo.health

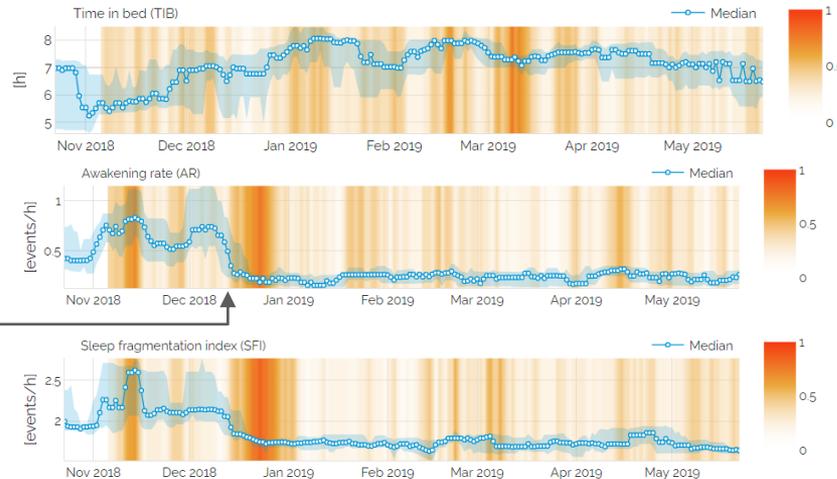EPFL Innovation Park, Building D
1015 Lausanne
Switzerland
T +41 58 800 58 00
www.domo-safety.com

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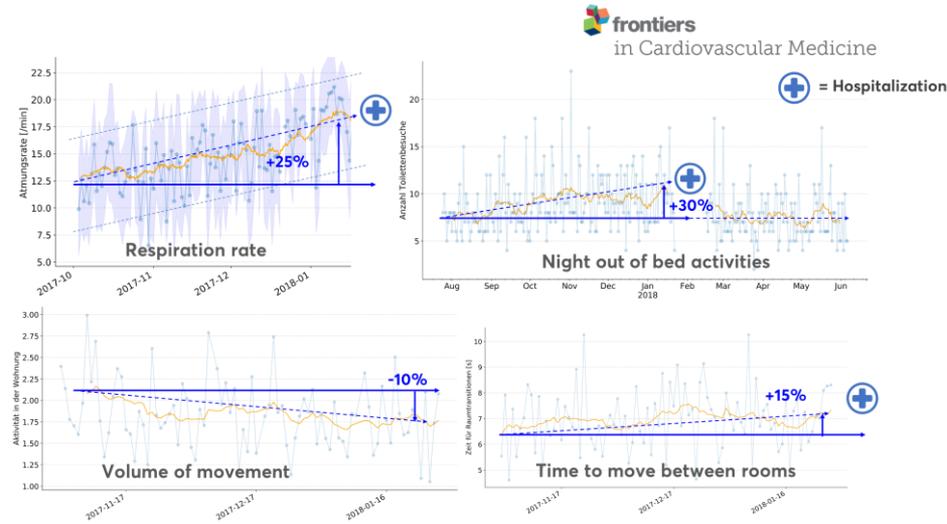
DOMO study in Switzerland to remotely monitor heart failure events

Study Rationale

- Heart failure (HF) is highly prevalent in western countries, including CH
- Before it becomes clinically obvious, asymptomatic structural and functional abnormalities may be captured by adequate monitoring
- Timely interventions may help avoid disease relapses & hospital admissions

Study Aim

Develop AI-based predictive models for early detection of acute heart failure decompensation based on remotely collected data from wireless and non-invasive sensors at home

Data collection

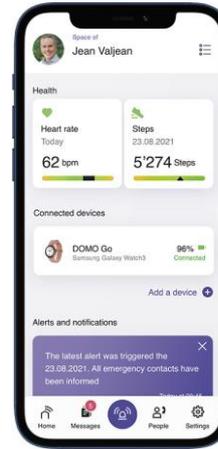
empatica 

- Pulse rate variability
- EDA
- Skin temperature
- Pulse rate
- Respiratory rate
- Blood volume pulse (BVP)
- 3-axis acceleration signal (activities...)
- Movement intensity

- Heart rate
- Respiratory rate
- HRV
- Bed occupancy
- Toss and turn
- Sleep efficiency

EMFIT™

domo.health 

- Ecological momentary assessments of self-reported fatigue
 - EQ-5D
 - KCCQ-12
- Age
- Sex
- Disease duration
- Treatment history
- Past Medical History

Multi-center study

Cardiology Department
Principal Investigator:
Dr Patrick Yerly

100 patients during 1 year

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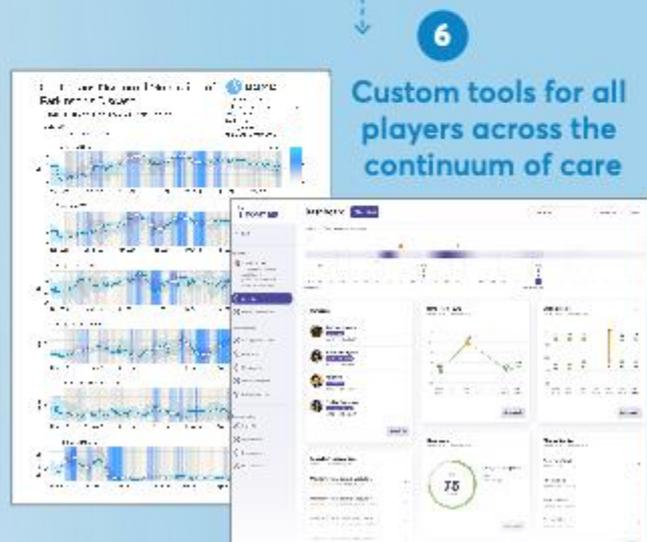
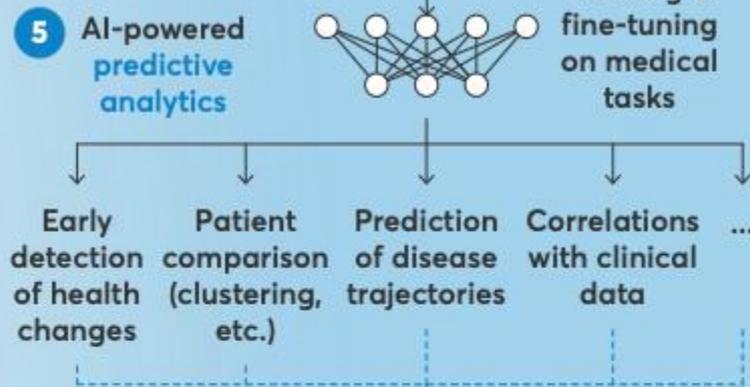
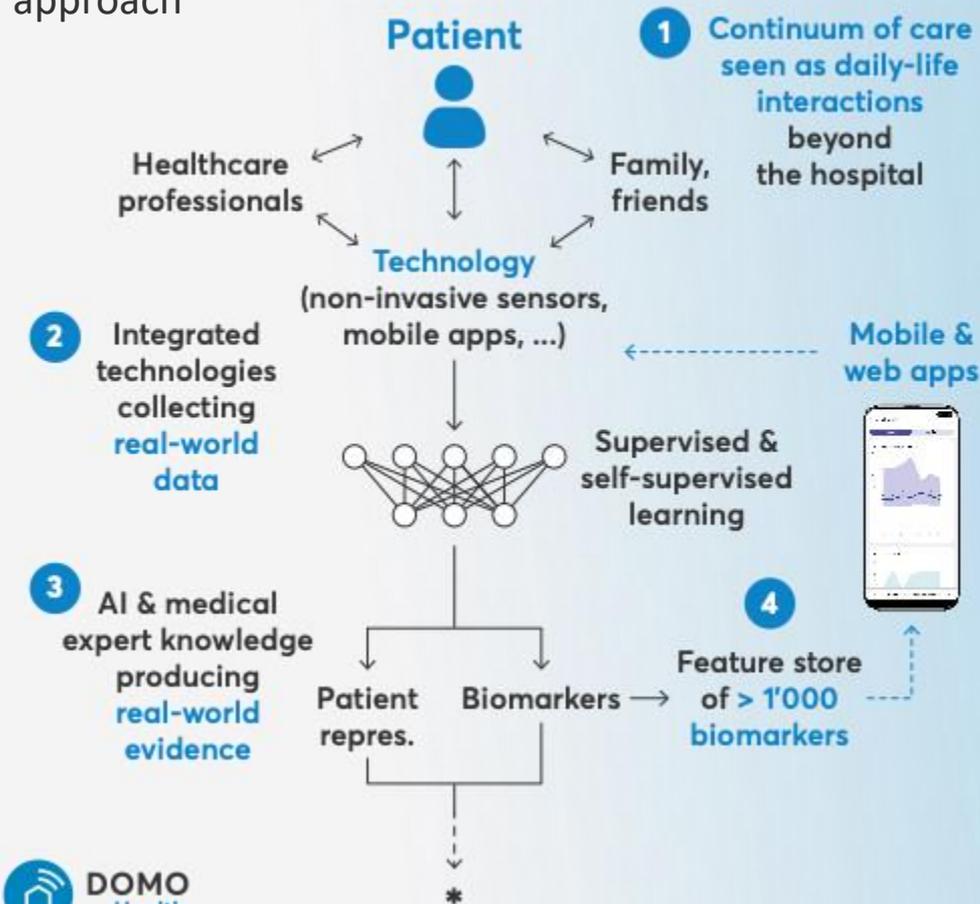
Cardiology Department
Co-Investigator:
Dr Grégoire Girod

*Patients will be **recruited directly at the hospital** following an episode of cardiac decompensation prior to hospital discharge

A patient-centered AI approach

Selected active collaborations

Heart failure
Early detection
during sleep

Spinal muscular atrophy
Detection of emergency
situations & respiratory
issues

Digital behaviorome
Development of biomarkers
for Alzheimer's disease,
aging, etc.

Self-supervised learning
Longitudinal, multiscale,
quantum-physics-inspired,
non-contrastive learning

Parkinson's disease
Patient-centered
monitoring

Hospital-at-Home
& other custom-made
solutions

Prof. Philippe Ryvlin
Head of the
Dept. of Clinical
Neurosciences

Dr. Julien Bally
Movement
Disorder
Specialist

Nicole Gusset
President of
SMA Europe

Prof. Dr. Tobias Nef
Head of the Technical
Group at the ARTORG
Center

Prof. Martin Jaggi
Head of the Machine
Learning & Optimization
Laboratory

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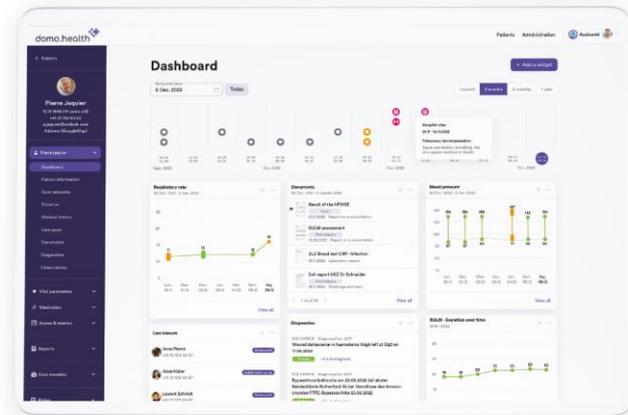
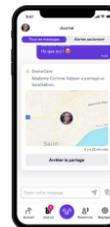
Croix-Rouge suisse
Schweizerisches Rotes Kreuz
Croce Rossa Svizzera

Your end-to-end digital health platform

domo.health

Prevention

Diagnosis

Treatment

Monitoring

domo.health app patient app

Health data visualization /
alert system 24/7

domo.health PRO platform

Integrated care / medication plan

domo.health PRO app

Patient management /
remote coordination

domo.health app patient app

Medication plan /
feedback in real-time

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Transmission

Facturation

The screenshot shows the domo.health PRO dashboard for patient Pierre Jaquier. The interface includes a sidebar with navigation options like 'Tableau de bord', 'Patient information', and 'Vital parameters'. The main dashboard features several widgets: a 'Respiratory rate' line chart, a 'Documents' list with items like 'Result of the HFMRSE' and 'RULM assessment', a 'Blood pressure' line chart, and a 'Réseau de soins' section listing caregivers like Aasiya Jaywant and Shichara Byoma. A 'Hospital stay' alert is visible, indicating a stay from 01.11 to 16.11.2022 with a diagnosis of 'Pulmonary decompensation'.

Hospital stay
01.11 - 16.11.2022

Pulmonary decompensation
Rapid and shallow breathing, the skin appear mottled or bluish.

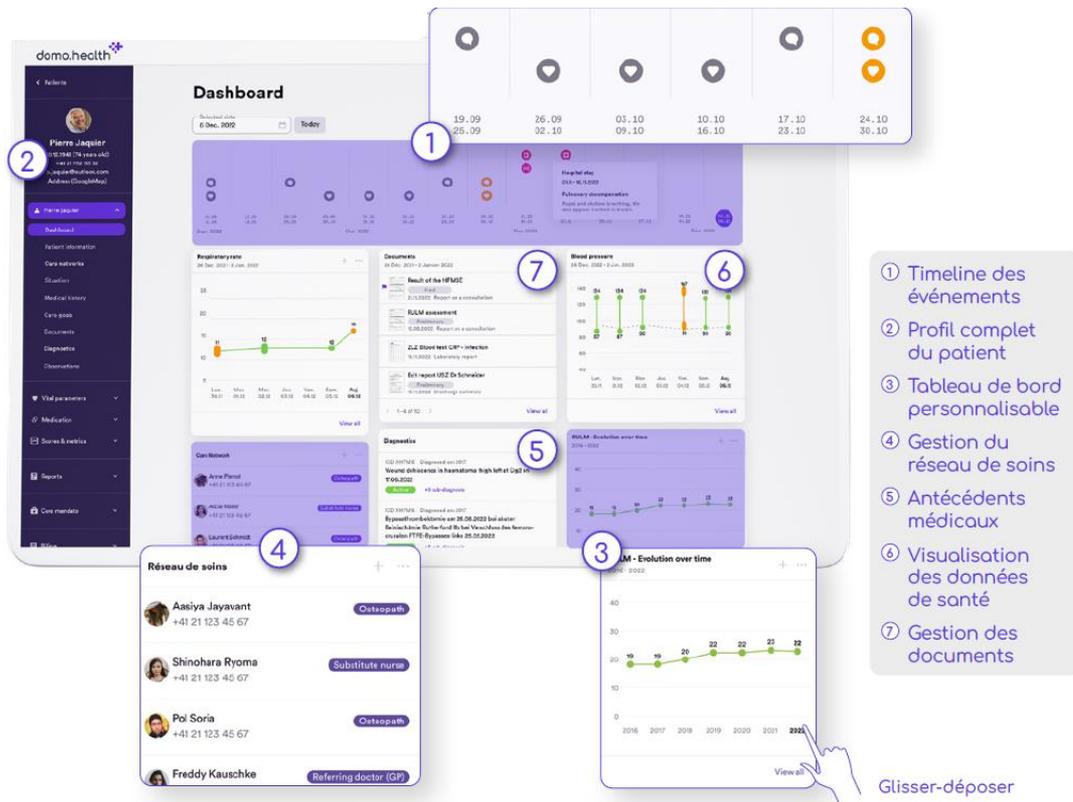
The Documents widget displays a list of medical reports and assessments, including 'Result of the HFMRSE', 'RULM assessment', 'ZLZ Blood test CRP - infection', and 'Exit report USZ Dr Schneider'. Each entry includes a date and a 'View all' link.

The Allergies & Intolerances widget lists substances such as 'X-ray contrast agent' and 'Histamine', along with associated symptoms like 'Rashes, itchy skin, headaches, or nausea' and 'Skin Redness, itching, swelling'.

The Sleep score widget shows an average sleep score of 75 out of 100, based on 7 values. It includes a circular progress indicator and a 'View all' link.

The RULM - Evolution over time widget displays a line chart showing the patient's RULM score from 2016 to 2022, with values ranging from 19 to 22.

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Today, 21/07/2022

Ringerfundin 1000ml
Ringerfundin
1000ml/24hrs ml/24hr After meal as infusion

Novalgin
Analgetikum, Metamizol
4g/24hr g/24hrs After meal as infusion

Tramal Inj L6s
Opioid-Analgetikum
300mg/24hr mg/24hr After meal as infusion

Dilution informations:
Novalgin (4g) + Tramal(300mg) / Ringer Lactat(1000ml)

Comment
Please strictly refer to the plan.

Add new medication

Medication

Administration routes

Per oral

Medication

EVRYSDI pdr 0.75 mg/ml

EVRYSDI pdr 0.75 mg/ml
Rudolph 0.75 mg
Duplicate Allergy Interference

Posology

Treatment Date / Time

Start Date: 30/06/2022 Start Time: 07:00 End Date: optional End Time: optional

Intake type

Scheme Periodic Daily Weekly Free Plan Extra Reserve

Posology according to scheme

Morning Noon Evening Night Use: Pcs Duration: optional Min

Comment

After meal.

Reason for use

Improve condition after intervention

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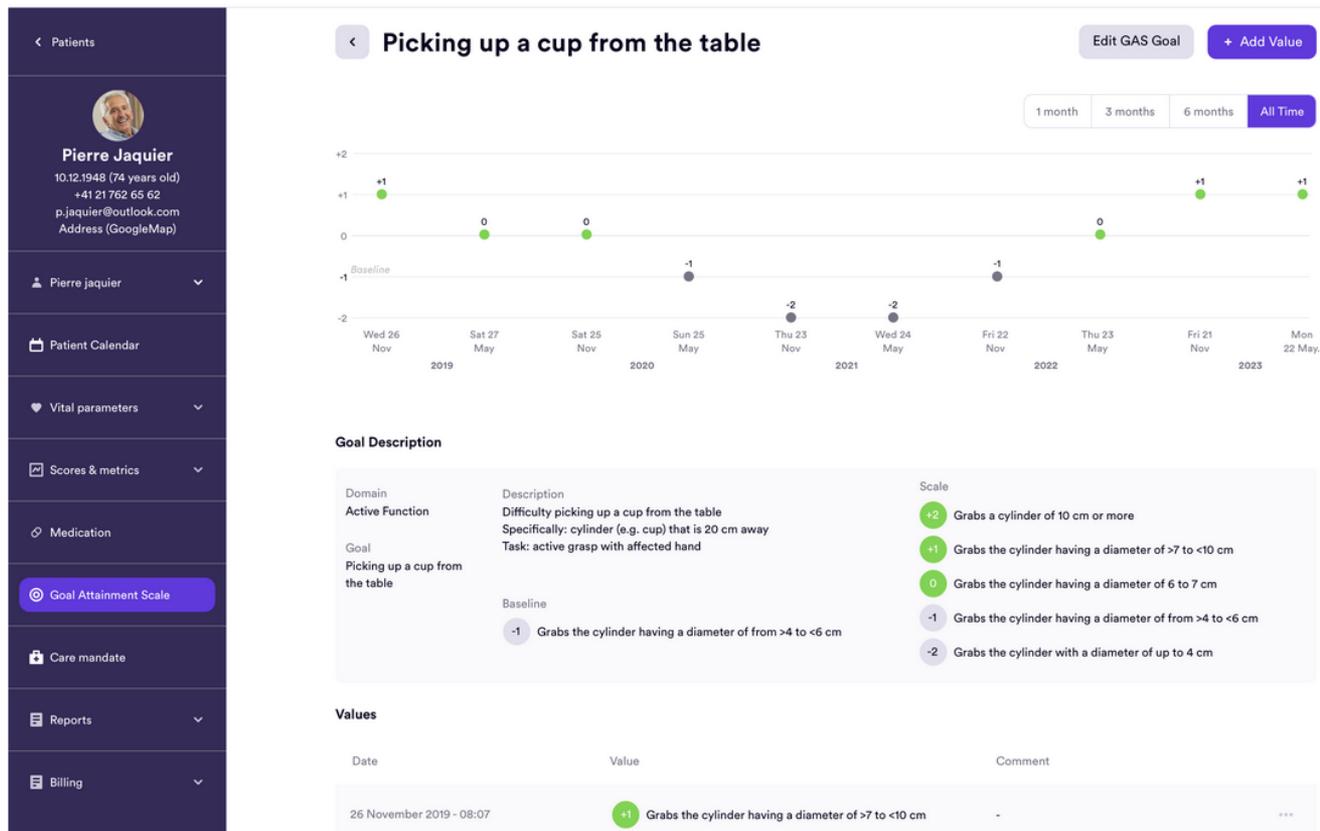
Lausanne, 3 May 2022

Medical report on the progress of treatment for Evrydsi

Pierre Jaquier, Ch. des Tilleul 25, 1032 Romanel-s-Lausanne, 546-215, 18/12/1948, 756.xxxx.xxxx.xx

File Edit View Insert Format

Paragraph B I

Progression of the disease and current status :
Slow progression. Currently amyotrophic flaccid tetraplegia; residual movements at distal and intermediate level, in upper limbs (M3). Bulbar involvement with fasciculation of the tongue. The patient is professionally active, as a teacher (employed by the State of Vaud, school sector), and uses an electric wheelchair.

Patient-specific therapeutic need:
Mrs Fernandez has scoliosis (treated surgically) and bulbar involvement as part of her illness. For these reasons, Spinraza® (nusinersen) cannot be administered to this patient.

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Date	Score
26 Dec. 2021	21
25 Jan 2022	25
22 Feb 2022	22
25 Mar 2022	25
25 Apr 2022	25
25 May 2022	25
25 Jun 2022	25
25 Jul 2022	25
23 Aug 2022	23

Document Informations

Document Name
SMA Follow-Up

Place and Date
Place and Date
Lausanne, 3 May 2022

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Subject
Medical report on the progress of treatment for Evrydsi

Add Patient information
Patient
Pierre Jaquier - 18.12.1948 (457-212)

Recipient
Pierre Jaquier, Ch. des Tilleul 25, 1032 Romanel-s-Lausanne,
546-215, 18/12/1948, 756.xxxx.xxxx.xx

Footer information

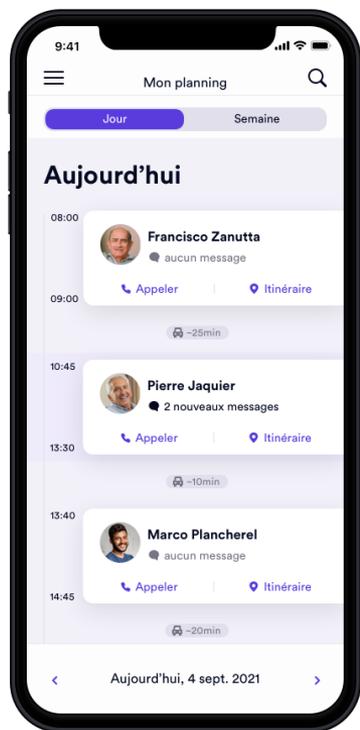
Author
Doctor Luc Ducommun

Signature
Author
Doctor Luc Ducommun *John Smith*

Author Info
Author info
Assistant doctor, Neuropsychology and neurorehabilitation
department

Footer
The presence of the names of the signatories attests to their
electronic validation of the document.

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domo.health[®] Nos compétences

Data Science, artificial intelligence

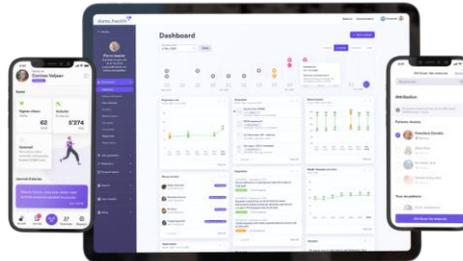
QMS compliant

Clinical study design and management

360° platform
patient centric
platform

Disease area expert knowledge and access to KOLs

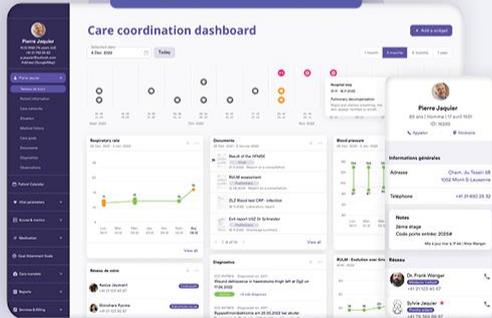
Medical certifications

AI-based platform empowering health professionals


Homecare

Streamline care with reduced admin, key monitoring, and seamless invoicing

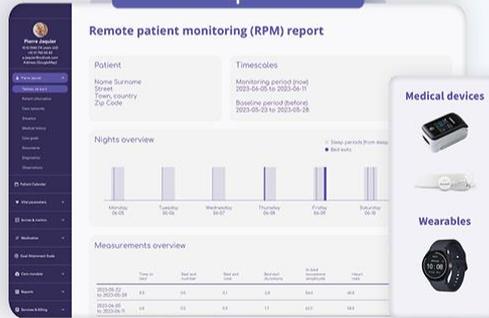
Hospital

Track vital signs and predict critical conditions remotely

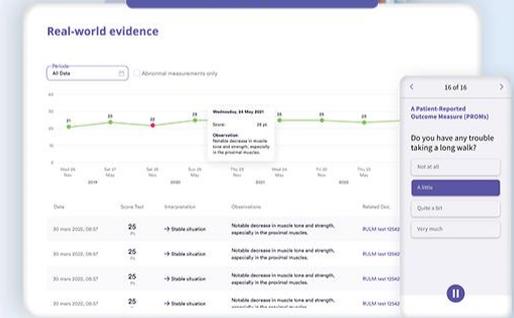Biotech

Enhance treatment outcomes for chronic conditions by harnessing real-world data

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